

KiteLock™ 4%

KiteLock™ 4% Sterile Catheter Lock Solution, 3mL

1.



Inspect the vials. Do not use it if it is leaking, the top is open, or the fluid is not clear in colour, cloudy, hazy or if there are any floating solid pieces in the fluid.

2.



Use an aseptic technique to open the vial (gloves on).

Tap the tab of vial and then wipe the tab and neck with an alcohol wipe before opening.

Holding the tab completely between your index and thumb twist off the top of the vial clockwise. Be careful not to touch the open part of the vial once open.

3.



Open a 10mL sterile syringe.

Hold the vial straight up.

Put the tip of the syringe into the open vial.

Twist the syringe onto the top of the open vial.

Turn the syringe clockwise until it is tight.

Holding onto the vial, pull slightly on the syringe to make sure it is firmly attached to the vial.

4.




Turn the syringe so it is under the vial.

Pull the syringe plunger down slowly to pull the fluid into the syringe.

When the vial is empty, hold the syringe plunger firmly so it doesn't get pulled back up.

5.

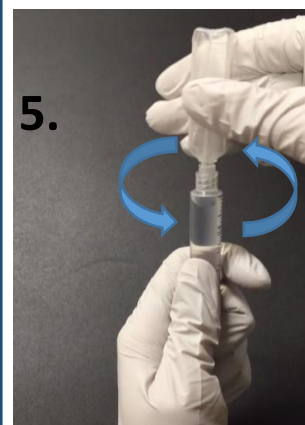

Turn the vial **counter-clockwise** and pull it off from the syringe.

Instill the necessary volume into the catheter as recommended by the manufacturer.

The catheter is now locked.

Remove KiteLock solution from the catheter before you use it. Twist a sterile empty syringe onto the catheter and pull back on the syringe to pull 2-3mL of fluid out of the catheter. Discard the syringe in a biohazard bin as it may contain blood.

Flush the catheter with normal saline to clear of blood. The catheter is now ready for treatment.