

# KiteLock™ 4%

## Sterile Catheter Lock Solution

### Instructions for use

## STEP 1

Visually inspect each vial prior to use. Do not use if vial is damaged, the seal is not intact or the solution is hazy, cloudy, discoloured or contains a precipitate.



## STEP 2

Flick the tab of vial to prevent excessive splashing of solution when opening.



## STEP 3

Using an aseptic technique, wipe the vial tab and neck with an alcohol wipe.



## STEP 4

Holding the tab between your index and thumb, twist off the tab of the vial in a clockwise motion (Do not touch the open part of the vial once open).



## STEP 5

To withdraw contents of vial, luer lock a 10 mL sterile syringe to the vial. While holding onto both the vial and syringe, invert and withdraw contents by pulling back on syringe plunger rod. Turn vial counterclockwise to remove from the syringe. Expel any excess air from the syringe.



## STEP 6

Attach syringe to Central Venous Access Device (CVAD) lumen and instill the required fill volume into the catheter (as per CVAD manufacturer recommendation).



#### Additional Considerations:

- KiteLock 4% should be aspirated, discarded, and replaced each time the CVAD is used.
- Follow hospital saline flush protocols prior to instilling and aspirating KiteLock 4%.
- KiteLock 4% is for single intravenous use only. One vial per CVAD lumen should be used. For multiple lumen CVAD, multiple vials are needed. Discard vial after use. Discard any unused portion. Do not reuse.
- KiteLock 4% Sterile Catheter Lock Solution may be used following the initial placement of a CVAD, after each injection of a medication, after each dialysis session or after withdrawal of blood for laboratory testing.

Refer to the KiteLock™ 4% DIRECTIONS FOR USE for full product details. STORE AT ROOM TEMPERATURE

WI-4 - KiteLock™ 4% - Instructions for Use  
Last Revised: February 12, 2021